

iChart Plus Installation Instructions

Place **Installation CD** into CD drive.

- Click “My Computer”
- Select your CD drive & double click
- Click “Set Up” icon
- Click “Next”
- EULA (End User License Agreement)
 - Select “I Agree” Click “Next”
- Default is “Install iChartPlus and Keyspan Remote Drivers” Click “Next”
- Keep default location of file. Change Default User to “Everyone” Click “Next”
- Click “Next”

iChartPlus Installation takes place.....

Keyspan Remote Installation

- Agree to EULA Click “Next”
- Accept Default file location Click “Next”
- **Change the “Restart my Computer Now” to “No, I will restart my computer later” Click “Finish”
- Installation is complete. Click “Close”
- Remove **Installation CD**

Installation is complete!

- Plug in USB remote receiver
- Remove plastic shield from battery on remote control
- Put **Installation CD** in a safe place.

***** Restart Computer Now *****

Insert **Runtime CD** and click **iChart Plus** icon

Calibrate to your office:

Measure distance from patient eye to computer screen in inches or meters (Direct View). (For Mirrored room, measure from eye to center of mirror and add distance of mirror to computer screen).

Click “H” on Keyboard to reveal “Radio Buttons”

Choose calibration screen

- Follow on-screen instructions to make letter correct size
- Name Calibration (ie: “Room 2 13.5 feet”)
- iChartPlus will save calibration on hard drive

Calibrating the iChartPlus for different office locations

Special Instructions for Dr. Vicky Fischer

- use above instructions to do calibrations at each office location & save the calibrations
- use instructions above to reveal the calibration page
- highlight previously calibrated setting
- Click “exit”
- Your screen will be calibrated to that particular office calibration setting until you change it

Note: The drilled hole found on the Runtime CD is intentional, not a defect.